



...Chemistry working for
Medicine & Dentistry...

Fissure Sealer and Bonding Agent Self-Cure

Catalog No. 50-01

CAT. NO. 50-01 (CLEAR KIT) CONTAINS:

50-011 Sealer, Part A (Clear) - 12.5cc
50-012 Sealer, Part B - 12.5cc
50-013 Enamel Conditioner - 13cc
30-30 Disposable Brushes & Handle
30-40 Mixing Wells
Instructions

CAT. NO. 51-01 (OPAQUE KIT) CONTAINS:

50-014 Sealer, Part A (Opaque) - 12.5cc
50-012 Sealer, Part B - 12.5cc
50-013 Enamel Conditioner - 13cc
30-30 Disposable Brushes & Handle
30-40 Mixing Wells
Instructions



CAUTION

Federal law restricts this device to sale by or on the order of a dentist.

This resin-based material is recommended for use as a Fissure Sealer in a caries prevention program. It may also be used as a Bonding Agent to improve adaptation of composite restoratives to cavity margins and to provide retention in "adhesive" Class IV restorations.

The Clear version is preferred for use as a Bonding Agent while the Opaque version is preferred for use as a Pit and Fissure Sealer.

Sci-Pharm's Fissure Sealer and Bonding Agent features low water absorption, assuring resistance to staining and durability in the oral environment. Its simple dispensing system makes application easy and economical.

SCIENTIFIC PHARMACEUTICALS, INC.
3221 PRODUCER WAY • POMONA, CA USA 91768
PHONE: (800) 634-3047 • (909) 595-9922
FAX: (909) 595-0331 • E-MAIL: scipharm@msn.com
WEBSITE: <http://www.scipharm.com>

Quality Management System Certified to
ISO 9001 and ISO 13485

CE Marked Products


Other Materials Available from Scientific Pharmaceuticals

<u>KIT</u>	<u>CONTENTS</u>	<u>MATERIAL CHARACTERISTICS</u>
CuRay-Fil Light-Cured Multi-Shade Composite Restorative Cat. No. 50-10	25g Restorative in 5 shades 6g Bonding Agent 13cc Enamel Conditioner Accessories & Instructions	Visible light-cure composite restorative for use in Class I, III, IV and V restorations featuring high polishability and x-ray opacity. Light-cure bonding agent included in the kit permits adhesive Class IV incisal edge restorations and assures excellent marginal seal.
CuRay-Match Light-Cure Paint-On Restorative and Veneer Cement Cat. No. 50-20	15g Restorative in 4 shades plus opaquer 6cc Porcelain Conditioner 13cc Enamel Conditioner Accessories & Instructions	A light-cure paint-on type composite for restoring eroded, hypocalcified, discolored or otherwise deficient human enamel as well as for cementing porcelain or acrylic veneers. Features include unrestricted working time, ease of handling and excellent aesthetics and durability. CuRay-Match is also sold in a 1.2g applicator syringe with disposable tips, offering convenient direct-to-the-tooth application. Sixteen shades are available.
Gel Etch, Enamel Etching Agent Cat. No. 70-12	12g of Etchant 25 Disposable Tips	A convenient to use, non-drying enamel etching agent offering excellent control of application.
Cure-On-Touch Adhesive Cat. No. 65-02	15g Adhesive 14g Primer 9g Enamel Conditioner Accessories & Instructions	The most popular and versatile type of orthodontic adhesive. Allows for bonding of both metal and plastic brackets.
Sci-Pharm Universal Varnish Cat. No. 70-03	15cc Varnish	A dentin sealer which is easy to apply and provides pulp protection against irritants from restoratives and cements. Minimizes post-operative tooth sensitivity and prevents marginal leakage. Compatible with all types of restorative materials and cements.
MicroJoin Crown & Bridge Adhesive/Cement Cat. No. 50-25	20g of Cement 15cc of Dentin Sealer 3g of Enamel Conditioner	A permanent cement for conventional and bonded restorations; suitable for use on vital and non-vital teeth.
CuRay-Eclipse Light-Cured Orthodontic Adhesive Cat. No. 60-05	20g Adhesive 7g Enamel Primer 13cc Enamel Conditioner Accessories & Instructions	Reliable and easy to use orthodontic bracket cement, offering virtually unlimited working time, long shelf-life and low sensitivity to storage conditions.

GENERAL INFORMATION

The use of fissure sealers to reduce the incidence of occlusal caries is well described in dental literature. Also, the use of liquid self-curing resins as primers for bonding composite restoratives to conditioned (etched) enamel has become a well-recognized and recommended procedure in dentistry. Application of bonding agent provides better marginal adaptation (thus reducing the incidence of secondary caries) and allows "adhesive" restorations which rely on bonding forces rather than on mechanical retention. Class IV restorations made of composite restoratives with liquid bonding agents as enamel primers have gained overwhelming acceptance among clinicians.

Sci-Pharm's Fissure Sealer and Bonding Agent represents an advanced formulation that is equally proficient in both applications. It features low shrinkage, high stain resistance, low water absorption, and a convenient application system.

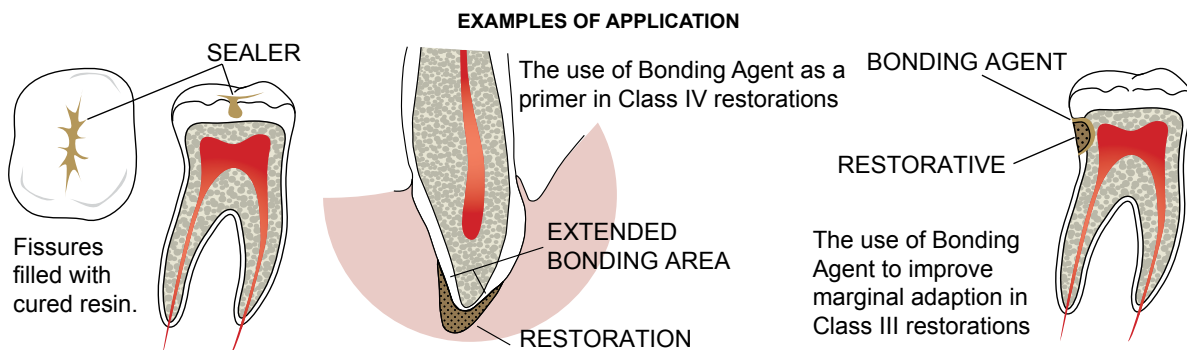

INSTRUCTIONS FOR USE

1. After prophylaxis, wash and dry the tooth (teeth) and apply Enamel Conditioner with a cotton pledget on enamel only. CAUTION: Avoid contact with soft tissue or exposed dentin. If accidental spill occurs, wash immediately.
2. Wait 30-45 seconds. Wash the tooth and evacuate. Dry with oil-free air or any commercial dental drying agent. Properly conditioned (etched) teeth should have a dull chalky appearance. Highly mineralized teeth may require an additional two-minute etch to achieve this effect.
3. Dispense an equal number of drops of Part A and Part B liquids into a mixing well and stir immediately with a disposable brush. Apply a thin layer of this mix over the dry, etched enamel. If the material is used as a fissure sealer, apply enough of the material to fill the fissure without, however, interfering with the occlusion. Working time is approximately 90 seconds from the start of mixing. The sealer will set in approximately 2.5 minutes. IMPORTANT: Maintain a dry field at all times during application and until the material sets.
4. A) When material is used as Fissure Sealer: After the Sealer sets, wipe the surface with a cotton pledget in order to remove the very thin film of uncured material remaining on the surface (the result of oxygen inhibition). Rinse and check the occlusion.
B) When material is used as Bonding Agent: Follow immediately by inserting the composite restorative. Do not wipe off the uncured surface layer of Bonding Agent.

HELPFUL HINTS

1. Re-examination every six months after application of Fissure Sealer is recommended. In order to reapply, polish the tooth with a sanding disc and apply Sealer following the procedure described above.
2. When sealing fissures, preferably work with one quadrant at a time.
3. For esthetic reasons, the clear version is the recommended choice for use as a Bonding Agent.

STORAGE AND SHELF-LIFE

Store at temperatures not exceeding 73°F (23°C). When stored under such conditions, the material has a shelf-life of 2 years. Refrigerate when the material is not in use (for example, overnight and on weekends). When cold, the material has a thicker consistency. For easier handling, remove from refrigerator at least 15 minutes prior to use.