

# StikFree<sup>™</sup>

## Releasing Agent

Cat. No. 70-05/71-05

**CONTENTS:** Cat. No. 70-05 - 15cc *StikFree*<sup>™</sup>  
Cat. No. 71-05 - 6cc *StikFree*<sup>™</sup>

*StikFree*<sup>™</sup> is an original formulation developed specifically to facilitate application of dental restoratives or cements which tend to stick to plastic and metal dental instruments. When used as directed, *StikFree*<sup>™</sup> will not alter or otherwise impair the properties or performance of these materials.

### **DIRECTIONS FOR USE**

Dispense one drop of *StikFree*<sup>™</sup> on a mixing pad. Wet all instrument surfaces that are likely to come into contact with the material. Remove any excess by gently scraping against the edge of the pad.

### **STORAGE AND SHELF-LIFE**

Store at temperatures not exceeding 75°F (24°C). When stored under such conditions, the material has a shelf-life of two years.

For technical information or to order, contact:  
Scientific Pharmaceuticals, Inc.

3221 Producer Way • Pomona, California, U.S.A. 91768  
Phone: (909) 595-9922 • (800) 634-3047 • Fax: (909) 595-0331  
E-Mail: [scipharm@msn.com](mailto:scipharm@msn.com) • Website: [www.scipharm.com](http://www.scipharm.com)

**Trust these trademarks...**

 **CURAY-FIL®**

**GLASION®**

**LUSTER®**

 **CURAY-MATCH®**

**TEMPO-REX® MICROJOIN®**

**CEMPER® DIAMOND LUSTER®**

 **CURAY-SUPPORT®**

**FloroPhos®**

**PCURE-ON-TOUCH L® UNTIL®**

**NEWCORE™**

**FLUOROSEAL®**

 **CURAY - PriMet®**

**FASTFIL®**

**SCI-SPAN®**

**LC SUPPORT®**

 **CURAY - II®**

**SUPPORT®**