



Manufactured in U.S.A. by:

**SCI-PHARM** 

Sci-Pharm Catalog No. 85-20  
TEETH WHITENER

**TECHNICAL BULLETIN**  
Instructions

*Sci-White*<sup>TM</sup>  
**Teeth Whitener**

Cat. No. 85-20

**FOR APPLICATION BY A PROFESSIONAL OR  
UNDER PROFESSIONAL SUPERVISION**

**KIT CONTAINS:**

- 12g Whitener in four syringes
- 12 Dispensing tips
- 2 Tray forming sheets
- Instructions



**OUTSTANDING FEATURES  
OF THE MATERIAL**

- Accelerated action = Shortened application time
- Low irritation potential = Reduced patient discomfort
- Pleasant mint flavor = Enhanced patient acceptance
- Amine free = Absence of questionable ingredients

**STORAGE AND SHELF LIFE:**

The shelf life of the material is six months when stored at temperatures not exceeding 73°F (23°C). Constant storage under refrigeration will extend the shelf life of the material to one year.

**INGREDIENTS**

Carbamide peroxide (16%) - an active ingredient, Glycerin, Buffered polyacrylic acid, Peppermint oil.

**CAUTION**

Federal law restricts this device to sale  
by or on the order of a dentist.

## GENERAL INFORMATION

Indications: Lightening of vital and nonvital teeth.

This product is not recommended for:

- Pregnant or lactating women.
- Heavy smokers or persons having habits of high alcohol consumption.
- Children below age 15 (although no significant adverse effects have been reported, little information has been published related to application of teeth whiteners on children).
- Users having inflamed or scarified oral tissues, leaking restorations or decayed teeth.

***Warning:*** Occasionally, patients may experience temporary discomfort in the teeth and gingiva. If discomfort persists, discontinue treatment.

## INSTRUCTIONS FOR USE

It is recommended that patients receive a prophylaxis and a dental examination prior to beginning the whitening treatment. A properly fabricated tray will improve the effectiveness of the whitening process and contribute to patient comfort. It is also important to explain proper application techniques to the patients to enhance the efficacy and comfort during home treatments.

### Treatment Procedures

1. Prior to first treatment, procure the trays and adjust to achieve a good fit.
2. Apply a small amount of tooth whitening material to the facial side of each tooth space within the tray.
3. Seat custom trays into place over the teeth. Do not squeeze the trays.
4. Remove any excess material from gingiva using a soft toothbrush, tissue, or finger.
5. The trays should be worn for 30-150 minutes, unless the dentist has instructed otherwise.
6. Remove the trays.
7. Brush teeth using a soft toothbrush and rinse mouth thoroughly.
8. Clean trays using a soft toothbrush and water.
9. Store trays in the tray case.
10. Repeat the procedure daily or as instructed by the dentist.

## ADDITIONAL NOTES

1. Do not eat, drink, or smoke while wearing the trays.
2. Keep out of reach of children
3. Wash spills immediately.

© Scientific Pharmaceuticals, Inc., 2006

For technical information, call or write:



CE  
0510

EU Authorized Representative:  
**QUALITY FIRST INTERNATIONAL LIMITED**  
20 Eversley Road, Bexhill-on-Sea, East Sussex TN40 1HE, UK  
Voice: + 44 20 8 522 1937 • Fax: + 44 20 8 522 1937  
E-mail: Hard6162@aol.com

**SCIENTIFIC PHARMACEUTICALS, INC.**  
3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768  
PHONE: (800) 634-3047 • (909) 595-9922  
FAX: (909) 595-0331 • E-MAIL: scipharm@msn.com  
WEBSITE: www.scipharm.com