



METAL PRIMER

Sci-Pharm Catalog No. 70-10

KIT CONTAINS:

- Primer, 15cc

INDICATIONS FOR USE:

A metal primer for in situ repair of damaged or worn veneers on cast restorations.

CONTRAINDICATIONS:

There are no contraindications.

CAUTION: Contains flammable solvent.

INSTRUCTIONS FOR USE

Using a fine brush, apply a thin layer of *CuRAY-PriMet*® on washed, roughened, and blow-dried exposed metal surface. Wait one minute. Cure for ten seconds with visible light curing instrument. Follow with the application of light-cured restorative.

HELPFUL HINTS

The use of paint-on (flowable) light-cured restoratives, such as Scientific Pharmaceuticals' *CuRAY-Match*®, is strongly preferred over conventional filling restoratives.

Extend the restoration over a relatively large area of porcelain ground down to feather edge around the area to be restored.

STORAGE AND SHELF-LIFE

Store at temperatures not exceeding 75°F (24°C). When stored under such conditions, the material has a shelf-life of two years.



For technical information, call or write:

Quality Management System Certified to

ISO 13485:2016



SCIENTIFIC PHARMACEUTICALS, INC.
3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768
PHONE: (800) 634-3047 • (909) 595-9922
FAX: (909) 595-0331 • E-MAIL: scipharm@msn.com
WEBSITE: <http://www.scipharm.com>

6079 - P1/2

Printed in U.S.A.

092624

...chemistry working for medicine and dentistry...

Trust these trademarks...

 **CURAY[®]-FIL[®]**

GLASION[®]

LUSTER[®]

 **CURAY[®]-MATCH[®]**

TEMPO-REX[®] MICROJOIN[®]

CEMPER[®] DIAMOND LUSTER[®]

 **CURAY[®]-SUPPORT[®]**

FloroPhos[®]

PCURE-ON-TOUCH[®] L[®] UNTIL[®]

NEWCORE[™]

FLUOROSEAL[®]

 **CURAY[®] - PriMet[®]**

FASTFIL[®]

SCI-SPAN[®]

LC SUPPORT[®]

 **CURAY[®] - II[®]**

SUPPORT[®]