

LUSTER®

Sci-Pharm Catalog No. 50-05

KIT CONTAINS:

- Paste, 20g (5 Syringes of 4g each)

CAUTION

Federal Law restricts this device to sale by or on the order of a dentist.

INDICATIONS FOR USE:

For use as a Dental Polishing paste for composite restoratives, amalgams and castings.

CONTRAINDICATIONS:

Not recommended for use on ceramics. If polishing ceramics, recommend using Sci-Pharm's *Diamond Luster®* Paste (Cat. No. 55-05).

OUTSTANDING FEATURES

- Absolutely no drying or changes in consistency
- Totally water soluble. Easily washes off teeth; cleans up and rinses off completely
- Cohesive and non-splattering; will stay where you want it
- Non-staining
- Convenient and cost effective method of dispensing

APPLICATION

Finish in conventional manner using your favorite stones, discs or very fine diamonds, and then polish with *Luster®* using prophyl cups.

STORAGE AND SHELF-LIFE

The shelf-life of *Luster®* is three years when stored at temperatures between 15°F (-10°C) and 83°F (30°C).



For technical information, call or write:

Quality Management System Certified to
ISO 13485:2016



SCIENTIFIC PHARMACEUTICALS, INC.
3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768
PHONE: (800) 634-3047 • (909) 595-9922
FAX: (909) 595-0331 • E-MAIL: scipharm@msn.com
WEBSITE: <http://www.scipharm.com>

6081 - P1/2

Printed in U.S.A.

092624

...chemistry working for medicine and dentistry...

Trust these trademarks...

 **CURE-RAY®-FIL®**

GLASION®

LUSTER®

 **CURE-RAY®-MATCH®**

TEMPO-REX® MICROJOIN®

CEMPER® DIAMOND LUSTER®

 **CURE-RAY®-SUPPORT®**

FloroPhos®

PCURE-ON-TOUCH L® UNTIL®

NEWCORE™

FLUOROSEAL®

 **CURE-RAY® - PriMet®**

FASTFIL®

SCI-SPAN®

LC SUPPORT®

 **CURE-RAY® - II®**

SUPPORT®