

DIAMOND LUSTER®

Diamond-Filled Polishing Paste for Dental Ceramics
(For Intra-Oral & Extra-Oral Use)
Sci-Pharm Catalog No. 55-05

KIT CONTAINS:

- Paste, 3g Syringe

INDICATIONS FOR USE:

For use as a Dental Polishing paste for ceramics.

CONTRAINDICATIONS:

Not recommended for use on composite restoratives, amalgams and castings. If polishing these, recommend using Sci-Pharm's *Luster®* Paste (Cat. No. 50-05).

Scientific Pharmaceuticals' *Diamond Luster®* is an excellent material for restoring original smoothness to dental ceramics after making repairs, adjustments, etc. Designed for both intra- and extra-oral use, *Diamond Luster®* is based on high purity, non-toxic ingredients. A polyglycol base of controlled molecular weight prevents splattering and facilitates wash-out without staining the tissue. The abrasive consists of pure diamond powder, whose particle size distribution has been selected for best efficiency and optimum gloss.

METHOD OF APPLICATION

The best way to apply *Diamond Luster®* is with felt polishing accessories.

STORAGE AND SHELF-LIFE

The shelf-life of *Diamond Luster®* is three years when stored at temperatures between 83°F (30°C) and 15°F (-10°C).



For technical information, call or write:

Quality Management System Certified to
ISO 13485:2016



SCIENTIFIC PHARMACEUTICALS, INC.
3221 PRODUCER WAY • POMONA, CALIFORNIA USA 91768
PHONE: (800) 634-3047 • (909) 595-9922
FAX: (909) 595-0331 • E-MAIL: scipham@msn.com
WEBSITE: <http://www.scipham.com>

Trust these trademarks...

 **CuRAY**  **-FIL**®

GLASION®

LUSTER®

 **CuRAY**  **-MATCH**®

TEMPO-REX® **MICROJOIN**®

CEMPER® **DIAMOND LUSTER**®

 **CuRAY**  **-SUPPORT**®

FloroPhos®

PCURE-ON-TOUCH **L**® **UNTIL**®

NEWCORE™

FLUOROSEAL®

 **CuRAY**  **- PriMet**®

FASTFIL®

SCI-SPAN® **LC SUPPORT**®

 **CuRAY**  **- II**®

SUPPORT®